Ermenrich Seek AT20 Alcohol Tester

EN User Manual

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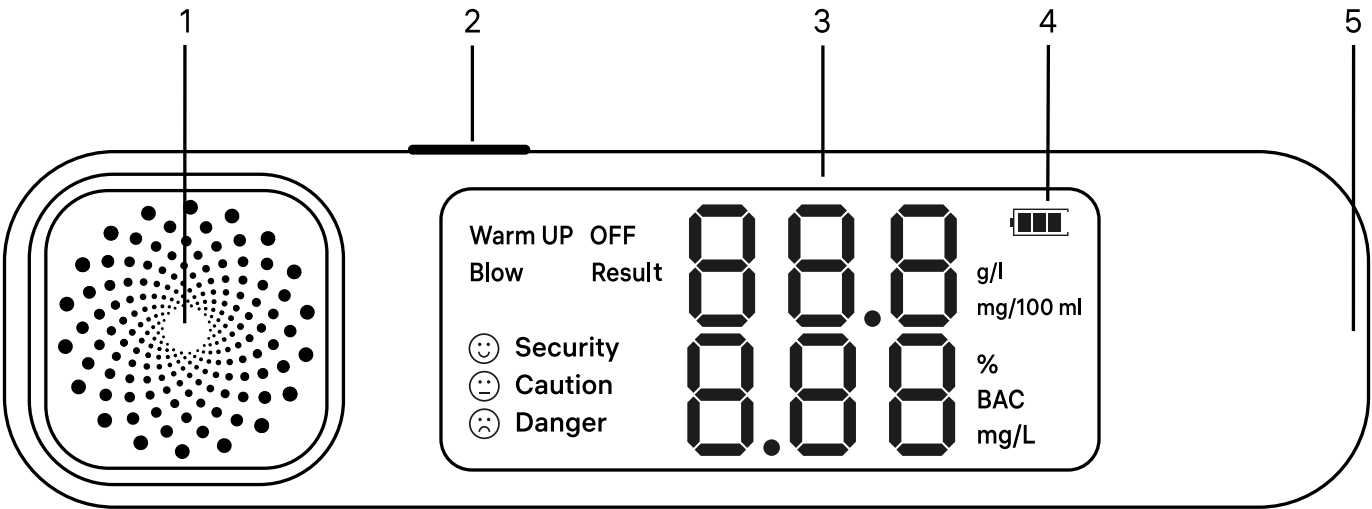
EN Ermenrich Seek AT20 Alcohol Tester

Please carefully read the safety instructions and the user manual before using this product. **Keep away from children.** Use the device only as specified in the user manual.

The kit includes: alcohol tester, Type-C USB cable, user manual, and warranty.

Charging the device

This device uses a rechargeable Li-Ion battery. Connect the power cable (included) to the device and the 5V, 1A DC adapter (not included) via a USB plug and connect it to the AC power supply to charge the battery.

Getting started

To turn the device on, press the Power button (2). **Warm UP** (Preparation time) will light up on the screen and the countdown from 10 to 00 will start.

■ If the alcohol tester sensor is used infrequently, it may oxidize. When using the device for the first time or after a long period of storage, perform several preparation cycles before conducting a test.

Conducting a test

Once preparation is complete, the device will automatically enter measurement mode. **Blow** (Response time) will light up on the screen and the countdown from 10 to 00 will start. Take a deep breath, bring the device close to your lips and exhale evenly until the time runs out. Stop exhaling after the beep. The screen will display the measurement results:

- At zero Blood Alcohol Content (BAC), there is no color indication on the display.
- At 0 to 0.03% BAC, the display will show **Security** (Safe level) in green.
- At 0.03 to 0.05% BAC, the display will show **Caution** (Alert level) in yellow and a warning tone will sound.
- For concentrations of 0.05% BAC, the screen will display **Danger** (Dangerous level) in red and a warning tone will sound.

When the measurement is complete, the device will turn off automatically.

■ The manufacturer is not responsible for illegal actions of the device user (drunk driving, working at heights, etc.).
■ The reaction to alcohol is individual, the test results are for reference only and cannot be used as an official conclusion.

Alcohol Content Correspondence Table

Blood Alcohol Content (BAC)				Breath Alcohol Content (BrAC)	
mg/100ml	%	‰	g/l	mg/l	ppm
10	0.01	0.1	0.1	0.05	25
20	0.02	0.2	0.2	0.10	50
30	0.03	0.3	0.3	0.15	75
40	0.04	0.4	0.4	0.20	100
50	0.05	0.5	0.5	0.25	125
60	0.06	0.6	0.6	0.30	150
70	0.07	0.7	0.7	0.35	175
80	0.08	0.8	0.8	0.40	200
90	0.09	0.9	0.9	0.45	225
100	0.10	1.0	1.0	0.50	250
110	0.11	1.1	1.1	0.55	275
120	0.12	1.2	1.2	0.60	300
130	0.13	1.3	1.3	0.65	325
140	0.14	1.4	1.4	0.70	350
150	0.15	1.5	1.5	0.75	375
160	0.16	1.6	1.6	0.80	400
170	0.17	1.7	1.7	0.85	425
180	0.18	1.8	1.8	0.90	450
190	0.19	1.9	1.9	0.95	475
200	0.20	2.0	2.0	1.00	500

- 1‰ = 0,1% BAC (Blood Alcohol Content)
- 1 g/l $\approx$ 1‰
- 1 mg/l in breath $\approx$ 2.1 g/l in blood
- 100 ppm $\approx$ 0.01% BAC
- 0.35‰ = 0.16 mg/l in exhaled air

Specifications

Measurement range	0–0.19% BAC (Blood Alcohol Content), 0–1.9 g/l
Preparation time	10s
Response time	10s
Power supply	3.7V, 200mA·h rechargeable Li-Ion battery
Operating temperature range	–10... +60°C (+14... 140°F)
Storage temperature range	–10... +60°C (+14... 140°F)

The manufacturer reserves the right to make changes to the product range and specifications without prior notice.

Care and maintenance

Do not try to disassemble the device on your own for any reason. For repairs and cleaning of any kind, please contact your local specialized service center. Protect the device from sudden impact and excessive mechanical force. Store the device in a dry cool place. Only use accessories and spare parts for this device that comply with the technical specifications. Never attempt to operate a damaged device or a device with damaged electrical parts! If a part of the device or battery is swallowed, seek medical attention immediately.

Battery safety instructions

The device is equipped with a built-in rechargeable lithium-ion battery. This avoids frequent battery replacement. Always switch the device off when not in use. If the battery charge is low, please recharge the device in time to ensure that the measurements are accurate. Do not overheat the battery. Do not discharge the battery completely.

Ermenrich Warranty

Ermenrich products, except for their accessories, carry a **5-year warranty** against defects in materials and workmanship. All Ermenrich accessories are warranted to be free of defects in materials and workmanship for **six months** from the purchase date. The warranty entitles you to the free repair or replacement of the Ermenrich product in any country where a Levenhuk office is located if all the warranty conditions are met.
 For further details, please visit: ermenrich.com
 If warranty problems arise, or if you need assistance in using your product, contact the local Levenhuk branch.

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